

Game Ready Control Unit



Next-Generation Game Ready System

New Game Ready Control Unit features wide reservoir opening, ergonomic handle, simple user interface, and expanded operation modes.

► [Order Now](#)

Where elastic bandages, messy ice packs, and other alternatives offer limited adjustability, the Game Ready™ System allows you to easily adjust pressure, manage temperature, and select treatment time settings—with just the push of a few buttons and the twist of a dial. Now you can simply and simultaneously apply intermittent compression and adjustable cold therapy to deliver the two most difficult-to-manage aspects of the RICE regimen in one easy-to-use treatment system.

Game Ready makes it easy.

To Order, contact:
MEDELCO Inc.
sales@medelco.ca
www.medelco.ca
Toll Free 1.800.268.7927

.... /continued next page.

❖ step 1



Fill the Control Unit with ice and water.

❖ step 2



Connect the Control Unit to the AC Adapter, then to a grounded electrical outlet.

❖ step 3



Apply the selected Wrap.

❖ step 4



Attach the Connector Hose to the Wrap, then to the Control Unit.

❖ step 5



Press the power button.

❖ step 6



Set the temperature to the coldest setting. After two minutes, adjust as needed.

❖ step 7



Select pressure and time settings in Manual Mode or select a program in Program Mode.

❖ step 8



Press the play/pause button to start.

The microprocessor-enabled Control Unit features...

Adjustable Compression. Three settings of cyclical pneumatic (air) compression or the option of no compression:

- High (5 to 75 mm Hg): approximately 2 to 3 minutes of inflation and 1 minute of deflation
- Medium (5 to 50 mm Hg): approximately 2 to 3 minutes of inflation and 1 minute of deflation
- Low (5 to 15 mm Hg): approximately 4 to 5 minutes of inflation and 1 minute of deflation
- No compression



Portability. The Game Ready Control Unit measures just 5.5"W x 16"L x 8.5"H and weighs only 8 lbs. empty. Easy to transport, it allows you to administer treatment on the go or wherever it is required. Optional Carry Bag makes traveling with Game Ready even more convenient.

Impact Resistance. Made from impact resistant plastic for durability.

Care and Storage. The Game Ready Control Unit can be cleaned with a mild disinfectant (warm

Temperature Adjustment. For cold therapy between 35°F to 50°F (when full of ice). Unit continuously cycles liquid through circumferential Wraps, even over larger surface areas.



Digital Timer. For convenient management of treatment time, without having to continually monitor the system:

- Timer can be set for a specific treatment time, up to 95 minutes; Control Unit automatically shuts off once the treatment time has elapsed.
- Timer also can be set to run continuously for up to 190 minutes.

water with mild detergent, Lysol, 70% isopropyl alcohol, or hydrogen peroxide). Store in optional Carry Bag or other protected place. Be sure to remove water and ice and dry before storing.

Optional Accessories

Connector Hoses



Carry Bag



Game Ready Control Unit

- ▶ Quick Start Guides
- ▶ User Manuals
- ▶ Case Study: Occupational Health
- ▶ Power of Recovery Brochure
- ▶ New Way to Heal Brochure
- ▶ Hand Wrap Brochure
- ▶ Order Now

Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

Warning: The purchase of a Game Ready™ System from a third party website, such as a public auction site, or through any private third party individual should be approached with caution. Game Ready is a prescription device whose use is regulated by U.S. Food and Drug Administration and state and local regulatory agencies. Sellers are required to sell such devices only to individuals qualified to purchase prescription devices (eg, health care professionals). While sellers are also required to confirm the condition of such devices, because there is no way to verify the condition of devices purchased from any of those sources identified above, the warranty for devices purchased from such sites or parties cannot be honored and will be void.

Original Control Unit

- ▶ Quick Start Guides
- ▶ User Manuals

